Operationsanleitung  
*Surgical Technique*

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Achtung: Operationsanleitung beachten  
 Attention: Be aware of the manual of surgical technique

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### Abbreviations:

|         |                     |
|---------|---------------------|
| AO-QC   | AO-Quick Coupling   |
| Mini-QC | Mini-Quick Coupling |
| WS      | Wrench Size         |
| as      | angle stable        |
| nas     | not angle stable    |
| av      | polyaxial           |
| st      | self-tapping        |
| mis     | minimal invasive    |
| -S      | Sterile             |
| -T      | Trial               |

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### Warning

This description does not suffice for immediate usage of the implants and instruments.  
 A briefing by a surgeon experienced in this field in the handling of these instruments is highly recommended.

|                                                                                                                            |             |
|----------------------------------------------------------------------------------------------------------------------------|-------------|
| <b>Vorwort</b><br><i>Preface</i>                                                                                           | <b>UA-1</b> |
| <b>Einleitung</b><br><i>Introduction</i>                                                                                   | <b>UA-2</b> |
| <b>Implantatspezifische Informationen</b><br><i>Implant specific information</i>                                           | <b>UA-3</b> |
| Indikationen<br><i>Indications</i>                                                                                         | UA-3.1      |
| Patientenlagerung und Zugänge<br><i>Plate specific details</i>                                                             | UA-3.2      |
| Plattenspezifische Details<br><i>Patient positioning and approaches</i>                                                    | UA-3.3      |
| Kontraindikationen<br><i>Contraindications</i>                                                                             | UA-3.4      |
| Implantatspezifische Informationen<br><i>Implant specific information</i>                                                  | UA-3.5      |
| <b>Allgemeine Informationen</b><br><i>General information</i>                                                              | <b>UA-4</b> |
| <b>Die richtige Lage der INTEOS® 2,5 Coronoid Platte</b><br><i>Correct positioning of the INTEOS® 2.5 coronoids plates</i> | <b>UA-5</b> |
| <b>Interoperative Handhabung</b><br><i>Intraoperative handlings</i>                                                        | <b>UA-6</b> |
| <b>Nachwort</b><br><i>Epilogue</i>                                                                                         | <b>UA-7</b> |

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*This document provides information about handling of Hofer Medical Solutions implants and instruments.*

*This manual of surgical technique is an addition and must not be considered as a substitute to existing literature about surgical methods within orthopaedics and traumatology.*

*The content shall be regarded as a recommendation for a standardized procedure of how to apply the products without addressing the issues of any further required measures, additional surgical interventions and possible extensions of the surgical technique.*

*The actual selection of the most suitable implant and its implantation method must happen exclusively by the surgeon based on his education and the individual diagnostic findings.*

*All illustrations printed here have a purely symbolic character to support the description of the surgical technique and can vary.*

*This surgical instruction doesn't contain any details on the use of the instruments. Corresponding documents are available in the form of*

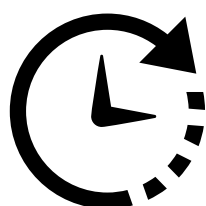
- *instructions for use for instruments: intra- and postoperative handling*
- *instructions for use for implants (is enclosed with each implant)*

*Please note that both the diagnosis and the determination of the treatment strategy rest solely with the surgeon.*

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*Hofer Medical Solutions gladly offers detailed training in safe handling and various surgical techniques.*

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Tel.: +43 (0)3382 53388

Mail: [office@hofer-medical.com](mailto:office@hofer-medical.com)

**Please contact our 24/7 service hotline:**

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*The INTEOS® Small Fragment System 2.5 from Hofer GmbH & Co KG is a multidirectional and fixed-angle Small Fragment Implant System based on the fixative internal concept.*

*This system generally serves to treat diaphysial, metaphysial and epiphysial fractures in small bones.*

*All Hofer Medical Solutions products are the result of collective development by experienced users and our own engineers. This successful collaboration leads to products that fulfil the anatomical and functional requirements of the relevant structure, from the easiest fractures to comminute fractures, on the basis of the anatomically shaped, low-profile designs as well as the various supply possibilities.*

**General advantages of the system:**

- *Minimised irritation of ligaments and soft tissue as a result of flat plate and screw geometry and rounded edges*
- *Lowest possible dimensions of plate thickness while increasing the plate stability through the smallest possible diameter of the „screw holes“*
- *Minimally protruding screw heads*
- *High angle flexibility of up to 70° when fully locked*

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## INTEOS® 2.5 Coronoid plate as av T5/1

## Indications

- Fractures of the coronoid at the proximal ulna

## Position of Patient and Approaches

## Patient Positioning:

- Standard

## Approach:

- Standard

## Plate Specific Details

- Perfectly anatomically pre-bent to increase the quality of the treatment and ease of use
- The extra flat and slim plate design allows a minimally invasive implantation, which is crucial for a quick restoration of the original mobility and function
- Should a bending nevertheless be necessary, this is possible at the discretion of the surgeon, between the screw holes
- Minimization of irritation due to low-profile design and completely retractable screw heads
- Angle stable and polyaxial locking mechanism
- The plate has a bendable flap to support and fix the tuberculum subliminus to restore elbow stability
- The plate geometry enables subchondral screw positioning for angular stable bridging of comminuted fractures
- K-wire holes for temporary plate and fragment fixation
- There is a hook on the proximal part of the plate to fix small fragments that are outside the screw range.
- Maximum variability for the best possible treatment, due to the compatibility with the screws and instruments of the INTEOS® 2.5 system

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#### Compatibility screws - Coronoid plate

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Compatible with S-INTEOS® 2.5 Coronoid Plate as vv

Screw diameter 1.5 mm  
Core diameter 1.1 mm  
Head diameter 3.0 mm  
angle stable & polyaxial

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Screw diameter 2.2 mm  
Core diameter 1.6 mm  
Head diameter 3 mm  
Drill bit for threaded hole 1.5 mm  
angle stable & polyaxial  
Compatible to ForceDRIVE T6

| Titan    | Durchmesser | Länge  | VPe |
|----------|-------------|--------|-----|
| Titanium | Diameter    | Length | PU  |

|                 |          |       |   |
|-----------------|----------|-------|---|
| 716-140-022-006 | Ø 2,2 mm | 6 mm  | 1 |
| 716-140-022-008 | Ø 2,2 mm | 8 mm  | 1 |
| 716-140-022-010 | Ø 2,2 mm | 10 mm | 1 |
| 716-140-022-012 | Ø 2,2 mm | 12 mm | 1 |
| 716-140-022-014 | Ø 2,2 mm | 14 mm | 1 |
| 716-140-022-016 | Ø 2,2 mm | 16 mm | 1 |
| 716-140-022-018 | Ø 2,2 mm | 18 mm | 1 |
| 716-140-022-020 | Ø 2,2 mm | 20 mm | 1 |
| 716-140-022-022 | Ø 2,2 mm | 22 mm | 1 |
| 716-140-022-024 | Ø 2,2 mm | 24 mm | 1 |
| 716-140-022-026 | Ø 2,2 mm | 26 mm | 1 |
| 716-140-022-028 | Ø 2,2 mm | 28 mm | 1 |
| 716-140-022-030 | Ø 2,2 mm | 30 mm | 1 |
| 716-140-022-032 | Ø 2,2 mm | 32 mm | 1 |

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Screw diameter 2.5 mm  
 Core diameter 1.9 mm  
 Head diameter 3.0 mm  
 Drill bit for threaded hole angle stable & polyaxial  
 Compatible to ForceDRIVE T6

| Titan           | Durchmesser     | Länge         | VPe       |
|-----------------|-----------------|---------------|-----------|
| <i>Titanium</i> | <i>Diameter</i> | <i>Length</i> | <i>PU</i> |
| 716-140-025-006 | Ø 2,5 mm        | 6 mm          | 1         |
| 716-140-025-008 | Ø 2,5 mm        | 8 mm          | 1         |
| 716-140-025-010 | Ø 2,5 mm        | 10 mm         | 1         |
| 716-140-025-012 | Ø 2,5 mm        | 12 mm         | 1         |
| 716-140-025-014 | Ø 2,5 mm        | 14 mm         | 1         |
| 716-140-025-016 | Ø 2,5 mm        | 16 mm         | 1         |
| 716-140-025-018 | Ø 2,5 mm        | 18 mm         | 1         |
| 716-140-025-020 | Ø 2,5 mm        | 20 mm         | 1         |
| 716-140-025-022 | Ø 2,5 mm        | 22 mm         | 1         |
| 716-140-025-024 | Ø 2,5 mm        | 24 mm         | 1         |
| 716-140-025-026 | Ø 2,5 mm        | 26 mm         | 1         |
| 716-140-025-028 | Ø 2,5 mm        | 28 mm         | 1         |
| 716-140-025-030 | Ø 2,5 mm        | 30 mm         | 1         |
| 716-140-025-032 | Ø 2,5 mm        | 32 mm         | 1         |
| 716-140-025-034 | Ø 2,5 mm        | 34 mm         | 1         |
| 716-140-025-036 | Ø 2,5 mm        | 36 mm         | 1         |
| 716-140-025-038 | Ø 2,5 mm        | 38 mm         | 1         |
| 716-140-025-040 | Ø 2,5 mm        | 40 mm         | 1         |
| 716-140-025-042 | Ø 2,5 mm        | 42 mm         | 1         |
| 716-140-025-044 | Ø 2,5 mm        | 44 mm         | 1         |

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Screw diameter 2.5 mm  
 Core diameter 1.9 mm  
 Head diameter 3.5 mm  
 Drill bit for threaded hole not angle stable  
 Compatible to ForceDRIVE T6

| Titan           | Durchmesser     | Länge         | VPe       |
|-----------------|-----------------|---------------|-----------|
| <i>Titanium</i> | <i>Diameter</i> | <i>Length</i> | <i>PU</i> |
| 716-145-025-020 | Ø 2,5 mm        | 20 mm         | 1         |
| 716-145-025-022 | Ø 2,5 mm        | 22 mm         | 1         |
| 716-145-025-024 | Ø 2,5 mm        | 24 mm         | 1         |
| 716-145-025-026 | Ø 2,5 mm        | 26 mm         | 1         |
| 716-145-025-028 | Ø 2,5 mm        | 28 mm         | 1         |
| 716-145-025-030 | Ø 2,5 mm        | 30 mm         | 1         |
| 716-145-025-032 | Ø 2,5 mm        | 32 mm         | 1         |

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### General instructions

- *The implant is anatomically shaped, however, should any adjustment be necessary, such adjustments can be made with the aid of the appropriate bending pliers*
- *The screws in the shaft area of the bone should always be locked bicortically*

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## **INTEOS® Coronoid plates**

*This special plate was developed for osteosynthesis of coronoid fractures.*

*The plate is anatomically pre-bent, should an adaptation of an individual shape of the bone be necessary, a rebending by using bending pliers is necessary.*

*The coronoid plates have k-wire holes for the temporary fixation at the desired position with k-wires of diameter of 1.2 mm.*

*The plate should be placed as proximally as possible so that the joint fragment can be fixed subchondral in the proximal row by using screws.*

*Another indicator of the correct positioning of the plate is a hook at the proximal end of the plate, which can be used to fix smaller fragments outside the screw range.*

*The bendable flap is used to support and fix the tuberculum subliminus to restore the initial elbow stability.*

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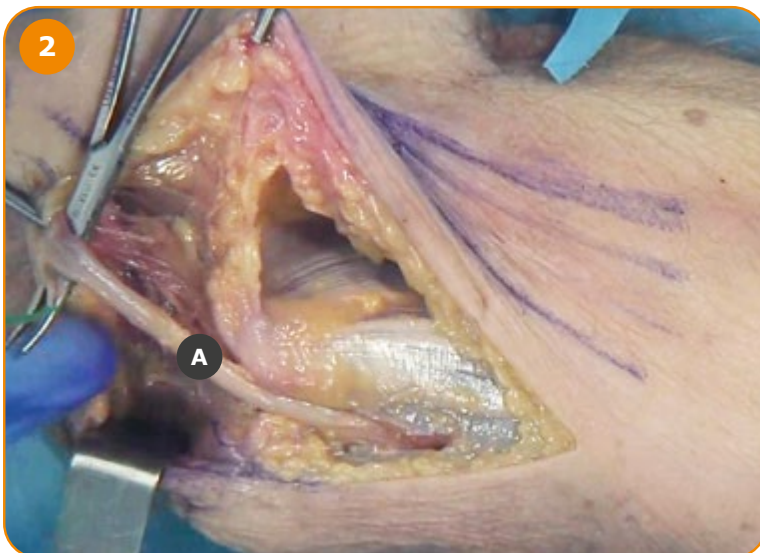
*The Hotchkiss "over the top" approach is the most anterior of the medial approaches and provides good access to the tip of the coronoid process and the whole anterior elbow joint.*

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*Identify the medial epicondyle **A**, the ulnar nerve **B** and the flexor carpi ulnaris **C**.*

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**Skin incision and Ulnar Nerve**  
*Make the skin incision posterior with a medial skin flap. Identify and protect the ulnar nerve **A**.*

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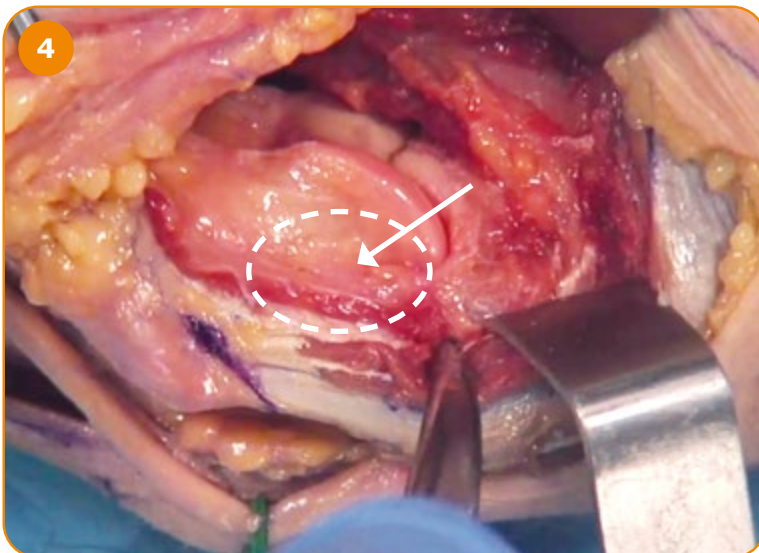
#### **Splitting the flexor-pronator mass**

Leave the space of one finger from the bottom limit of the flexor carpi ulnaris. Split the flexor-pronator mass and elevate the palmaris longus, flexor carpi radialis and pronator teres origins off the medial epicondyle.

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#### **Capsulotomy**

Identify the anterior part of the medial collateral ligament that lie under the muscle. Perform an anterior capsulotomy to enter the elbow joint.

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© 2023 Hofer GmbH & Co KG. All rights reserved.

*Operation instruction booklets, handbooks, information brochures and software are protected by copyright. Copying, duplication, translation or conversion of said document through any electronic means or machine readable form, whether that be in full or in part, is not permitted without prior written authorisation from Hofer Medical Solutions.*

*All further rights regarding the software are defined in the provided licensing terms.*

*The Hofer Medical Solutions logo is a trademark of HOFER GmbH & Co KG and is registered in Austria and additional countries.*

*The rights to other brand and product names mentioned in this scripture are reserved by their proprietors and are accepted and recognised here. The naming of products that are not Hofer Medical Solutions products serves solely as a source of information and not of advertising. Hofer Medical Solutions accepts no liability regarding the selection, performance or efficiency of said products.*

*The methods and practices, etc. quoted in this scripture have been carefully tested and verified. However, should errors or suggestions of improvement be apparent, despite said verification, we politely ask you to disclose them to us. The surgical method portrayed here merely presents one possible method for the indication that is to be treated. It remains the responsibility of the relevant surgeon to either keep to the aforementioned osteosynthetic method or to vary the method according to individual needs. Direct and indirect complications as well as sequential complications are thus, in every case, excluded.*

*Should you have any further questions, please do not hesitate to contact us at any time.*

#### **ÖSTERREICH**

HOFER GMBH & CO KG  
Jahnstrasse 10-12  
A 8280 Fürstenfeld  
TEL: +43(0)3382/53388  
FAX: +43(0)3382/53093  
office@hofer-medical.com  
www.hofer-medical.com  
FN21826y  
UID ATU30764704

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